


CARA PRINT COCKPIT

RELEASE NOTES

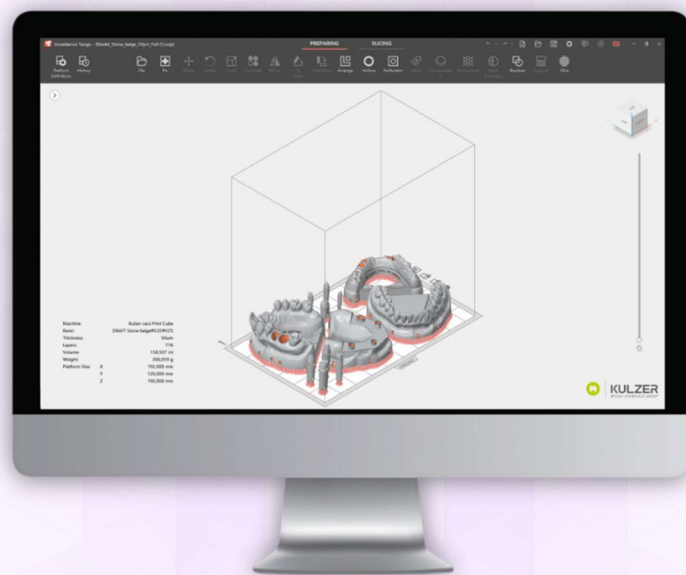

Version: 04
Date: 08.07.2026



[www.kulzer.com/
mycube-support](http://www.kulzer.com/mycube-support)



KULZER
MITSUBI CHEMICALS GROUP

General Information

The installer for cara Print Cockpit from Kulzer always contains the latest version of cara Print Cockpit AND the latest version of the Tango slicer.

Below, please see a historic list of cara Print Cockpit versions including the release date.

cara Print Cockpit version	Release date
V1.0.03	2024-11-15
V1.0.04	2025-01-31
V1.0.05	2025-04-22
V1.0.06	2025-12-04
V1.0.08*	2026-07-08

Below, please see a historic list of Tango versions including the release date.

Tango version	Release date
4.0.23.23	2024-11-15
4.0.33.1	2025-01-31
4.0.35.12	2025-04-22
4.0.57.21*	2026-07-08

*latest version

Revision History

cara Print Cockpit

cara Print Cockpit V1.0.04	
1	Check box added in the installation process to make it possible to install the software for all users on the same computer.
2	Information added to “restart the software” after having changed the language
3	Calculation and display of the expected printing time per print job
4	Sign added: Trusted installer with signature
5	Improved translations for all available languages
6	Left click on the Kulzer logo will lead to Kulzer’s support hub.

cara Print Cockpit V1.0.05	
1	“User defined Print Parameters” function added

cara Print Cockpit V1.0.06	
1	Automatic alert function added in cara Print Cockpit whenever a new firmware for the cara Print Cube is available
2	“Reminder window for z-compensation” added
3	.capc viewer added
4	Supports for dima Print Stone Teeth 2, dima Print Stone gray and dima Print cast emerald added
5	Japanese translation improved
6	Translations for “customized print parameter” function improved in all available languages

cara Print Cockpit V1.0.08*	
1	Spanish language added

Tango

Tango 4.0.33.1	
1	Portuguese and Italian translations available
2	Updated German translations
3	Auto save function for the Tango project (.vxp) possible. The program will auto save the project to a user-defined path when exporting the slicing file. Kulzer recommends to always have this function activated, as this can help the user and Kulzer’s support team, if necessary

Tango 4.0.35.12	
1	Japanese language translation added
2	German translation improved

Tango 4.0.57.21*	
1	Automatic z-height preset for every dima Print material
2	“One click” functionality for all dima Print model materials (automated support generation and nesting)
3	Spanish language added