Selekt+ Sterile Single-Use Dental Needles

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KULZER
MITSUBI CHEMICALS GROUP

Manufacturer:
SOFIC, B.P. 282
81207 Mazamet Cedex, France
www.sofic.com

- **SELEKT+ Carpule** – Sterile single-use stainless steel siliconised dental needles with bevel marking.

Indication
Injection of local dental anaesthetic.

Contraindication
Hypersensitivity to any component.

- Instructions for use**
- Break the sterility guarantee seal just before use by turning the two plastic caps in opposite directions.
 - Remove the rear cap
 - Screw the pre-threaded needle onto the pre-threaded hub of the syringe.
 - Remove the plastic cap just before the injection is administered.
 - After use, unscrew using a safe removing procedure complying with local practices/regulations.

Disposal
Discard the used needle into a puncture-resistant container in accordance with applicable local medical waste regulations.

- Warnings**
- In the event that the sterility guarantee seal is already broken, do not use the needle.
 - Never recap needles by hand. Recapping needles by hand or recapping bent or broken needles can cause unintended needle sticks.
 - Do not bend, or otherwise stress needles as serious injuries to you and/or your patient can occur.
 - Do not insert needle to hub during injections, as needles can break and become lodged in patient's tissue, potentially causing serious permanent injury.

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Distributor:
Kulzer GmbH
Leipziger Straße 2
63450 Hanau (Germany)

CE 2797 (CE: 12/1995)

- Instructions for use**
- Avoid applying excessive pressure and excessive needle movement during injection as this may cause needle breakage, which may result in serious injury to you and/or your patient.
 - The needle is a single-use device, do not reuse the needle on a different patient to avoid cross contamination.
 - Do not reuse the device with several cartridges to avoid performance issues (cannula penetration) and cannula breakage.
 - Do not re-sterilise otherwise sterility and performance of this device cannot be guaranteed.
 - Be especially vigilant when using a needle in a patient who appears to be apprehensive.
- Cautions**
- Store in original packaging until ready to use.
- Storage**
Do not store above 30°C.
- Presentation**
100 needles
- For professional dental use only**
The choice of the most appropriate cannula is the responsibility of the practitioner.
- Dated: 2020-06

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