KULZER
VIVIDA

**SHADE
CORRECTION**

25 % HYDROGEN PEROXIDE SUPERIOR

INSTRUCTIONS FOR USE

BG	Инструкции за употреба	3
CZ	Návod k použití	6
DE	Gebrauchsanweisung	9
DK	Brugervejledning	12
EE	Kasutusjuhend	15
EN	Instruction for Use	18
ES	Instrucciones de uso	21
FI	Käyttöohjeet	24
FR	Instructions d'utilisation	27
GR	Οδηγίες χρήσης	30
HR	Upute za uporabu	33
HU	Használati utasítás	36
IT	Istruzioni per l'uso	39
LT	Naudojimo instrukcija	42
LV	Lietošanas instrukcija	45
NL	Gebruiksaanwijzing	48
NO	Bruksanvisning	51
PL	Lietošanas instrukcija	54
PT	Instruções de uso	57
RO	Instrucţiuni de utilizare	60
SE	Bruksanvisning	63
SI	Navodila za uporabo	66
SK	Návod na použitie	69
TR	Kullanma talimatı	72

Kulzer Vivida Office Shade Correction

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EN Instruction for Use

Kulzer Vivida Office Shade Correction

Professional Shade Recovery Product

DESCRIPTION

Kulzer Vivida Office Shade Correction gel is based upon reactive oxygen which is added as stabilized encapsulated hydrogen peroxide. On contact with the discoloration on the enamel, the hydrogen peroxide breaks down rapidly into oxygen radicals and water. During the process the free radicals attack the double carbon bonds that are the source of most colors (stains) in the tooth.

INDICATIONS

Kulzer Vivida Office Shade Correction is a single use, simple in-office product used for shade corrections of vital and non-vital teeth in just 30–45 minutes.

CONTRAINDICATIONS

It is recommended to consult a doctor for:

- Pregnant and lactating women.
- Those currently treated for a serious illness.
- Children under the age of 18 years.

CONTENTS

- 3x 2ml Kulzer Vivida Office Pen(Office Starter Pack)
- 12x 2ml Kulzer Vivida Office Pen(Office Pen Refill)

Active Ingredient

- 25 % Hydrogen Peroxide

COMPOSITION

- PEG, water, hydrogen peroxide, poloxamer, t-butyl alcohol, etidronic acid, peppermint oil, EDTA, sodium stannate

CLINICAL PROCEDURE

Pre-treatment

- Review current dental health and make realistic expectations about final results.
- Provisional or temporary restorations may discolor during the whitening process while composite or polycarbonate materials will remain unaffected.
- Composite restorations may require replacement, since only natural teeth will whiten.
- For aesthetic or adhesive restorations, wait two weeks after the last whitening treatment to allow color stabilization.
- All restorations should be intact or well-sealed with adequate temporary materials.
- It is recommended to use a sensitivity relief product (i.e. Kulzer Vivida Tooth Conditioner) two weeks prior to, and two weeks after the whitening treatment.
- Determine the initial tooth shade with a photo.
- Polish the teeth with prophy angle and prophy paste to remove plaque.

Barrier procedure

- Insert a cheek retractor and dry the gingiva with air.
- Apply the Kulzer Vivida Gingiva barrier to the gingival tissue along the margins of the tooth according to the instruction for use of Kulzer Vivida Gingiva barrier.

Correction procedure

- Apply a thin layer of gel onto the teeth.
- Please use the dosing lines on the pen for reference.
- After ten to fifteen minutes remove the gel by suction.
- Repeat the application and removal steps two to three times.
- Rinse the teeth with water and suction.

Post-treatment procedure

- Determine the final tooth shade with a photo.
- Advise the patient to use a sensitivity relief agent (i.e. Kulzer Vivida Tooth Conditioner) for at least two weeks after treatment.

POTENTIAL ADVERSE EFFECTS

- Gel that comes into contact with the gingiva may cause severe soft tissue irritation.
- Remove immediately the gel of the gingiva by suction in case you notice it has touched the gingiva.
- Increased tooth sensitivity to cold or non-specific sensitivity in the teeth, gums, tongue, lips or throat can occur during the treatment. In general, symptoms should disappear within a few days after the treatment. The treatment can be proceeded with shorter, less frequent applications until

the desired whitening effect has been achieved. If the discomfort remains, discontinue the treatment.

DISPOSAL INFORMATION

- Dispose of device in accordance with local waste regulations.

WARNINGS AND SAFETY INSTRUCTIONS

- If gel is swallowed, immediately drink water, do not induce vomiting, and contact a doctor.
- If gel gets into eyes, hold eyelids apart and flush the eye continuously with running water for at least fifteen minutes and contact a doctor.
- If gel gets into contact with clothing, skin or hair contact occurs, remove contaminated clothing and flush skin or hair with running water.
- Contains hydrogen peroxide.
- Keep out of reach of children.

STORAGE CONDITION

- Store in a cool, dry place (frost-free) and protected from direct sunlight.
- Do not use the material after the expiry date.

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© 2025 Kulzer GmbH. All Rights Reserved.
Kulzer GmbH